Foot/Ankle
(Revised: 11/22/22)

ARABIC

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ENGLISH

Instructions for Use

1. Targeted vibration therapy aids in pain relief and recovery for your shoulders, back, arms, knee, calf, foot, ankle and more.
2. Three distinct levels of vibration offer customized relief.
3. Universally sized and designed to fit various body parts.
4. Snap HUB into therapy wrap, adjust strap to hold band in place with velcro closure.
5. To turn on, simply hold button on side of the band for 2 seconds until a blue light is illuminated.
6. Press button once to begin low frequency vibration. Press button again to toggle between desired vibration levels.
7. Apply vibration therapy for 25 - 45 minutes for maximum level of pain relief.
8. When finished, simply hold button down for 2 seconds to turn off.
9. Replenish battery by plugging USB charger into the charge port.
10. Spot clean as needed.
11. Lay flat and allow to air dry.
12. Keep charger opening dry while cleaning.
13. Do not dry clean.

Material: 82% Nylon / 18% Spandex Pile 100% Styrene-butadiene, Rubber Foam Core, 100% Nylon Back

© 2022 Brownmed - Intellinetix is a registered trademark of Brownmed (877) 853-5518 U.S. Patent #14/011,838

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© 2022 Brownmed - Intellinetix is a registered trademark of Brownmed (877) 853-5518 U.S. Patent #14/011,838
Rev. 0

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
© 2022 Brownmed - Intellinetix is a registered trademark of Brownmed (877) 853-5518 U.S. Patent #14/011,838

Rev. 0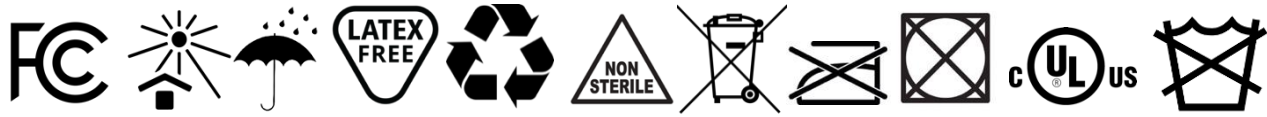

EC REP CE
CEpartner4U
Esdorrielaan 13
3951 DB Maarn
The Netherlands
www.cepartner4u.eu
Manufacturer:
Brownmed
1300 Lundberg Drive West
Spirit Lake, Iowa 51360 U.S.A.

UDI

MD

© 2022 Brownmed - Intellinetix is a registered trademark of Brownmed (877) 853-5518 U.S. Patent #14/011,838 Rev. 0

REF LOT